


User Manual

2nd Assi™

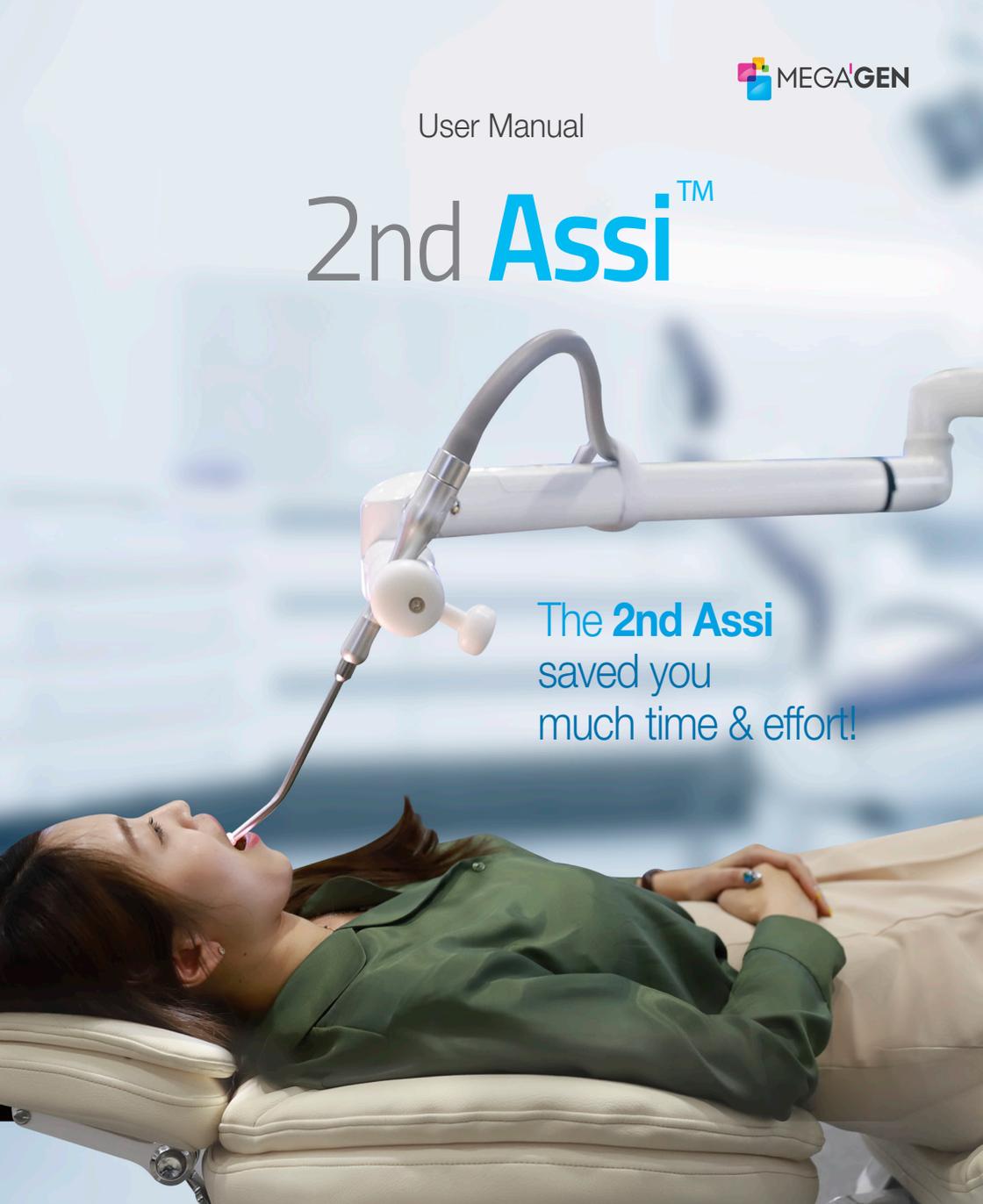

The **2nd Assi**
saved you
much time & effort!

INSTRUCTION MANUAL

Read before Installation and Usage.

 **MegaGen Implant Co.,Ltd** **CE**

45, Secheon-ro 7-gil, Dasa-eup, Dalseong-gun,
Deagu, Korea

Tel : 82-53-222-2828 / Fax : 82-53-289-3420



Contents

1. Concept	04
2. Part Introduction	04
3. How to use	06
4. How to use: Manual Movement	07
5. Precautions	11
6. Product Specifications	11

1. Concept


Thank you for purchasing the 2nd Assi.

Please read this manual thoroughly before using and following the instructions.

This equipment is intended to hold the suction instrument while treatment. The 2nd Assi helps the suction work perfectly, without shaking at the desired location.

2. Part Introduction


2nd Assi

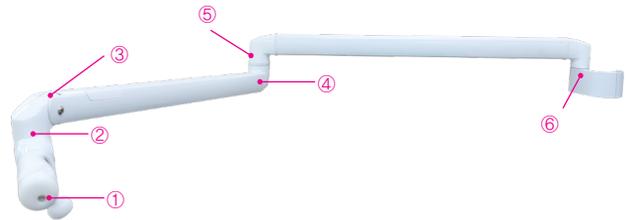

Suction holder attaching to chair's Light pole.


It consists of 6 joints with its holder.

Pole Holder

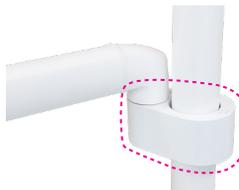


It holds '2nd Assi'. It can be attached to another type of pole with adjustable holding-grip and fixing screw.

6th joint

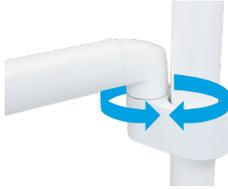


It can be rotated left right horizontally.

5th joint

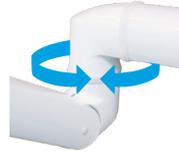

4th joint

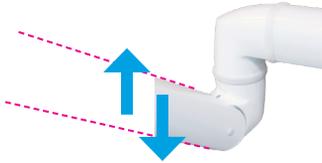

It can be moved up down vertically.

3rd joint



2nd joint



It can be rotated left right horizontally.

1st joint & Suction Holder



It rotates 360° degree.
Position(1): Holder for metal suction
Position(2): Holder for other type of suction

3. How to use

3.1 Preparation before use

1. The product should be used by person who goes through or has required necessary training, knowledge or experience.
2. The product should be used according to intended use.
3. The product should be used or stored after sanitization.
4. Always check for wear, cracks, deformation, corrosion, damaged parts before use.
5. Do not use the product if it is damaged.
6. Replace the damaged parts with spare parts immediately before use.

3.2 How to use

1. The product should be used after reading this user's manual.
2. Adjust the joints to get to the patient's oral position for treatment.

3.3 Storage & Maintenance after use

1. Make sure that there is no foreign substance left on the product after using it.
2. Make sure that you fold the product and store it in the place in which there is no moisture.

3. 4 How to clean


1. Alcohol cotton is recommended.
2. The product should be cleaned and dried after the contamination.
3. Dry the product by cloth or air after disinfection.
4. If necessary, repeat cleaning or disinfection.

4. How to use: Manual Movement


A. Place the suction on suction holder.




B. By using 6th joint, you can rotate it left right.

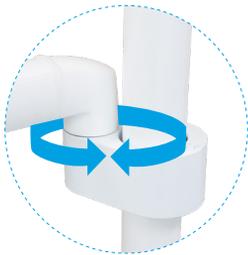


C. By using 5th joint, you can rotate it left right.

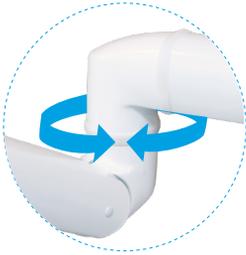

D. By using 4th joint, you can move it up down.

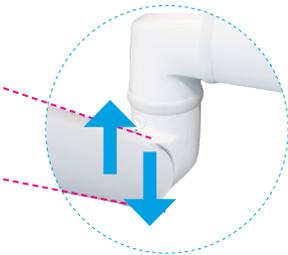


E. By using 3rd joint, you can move it up down.

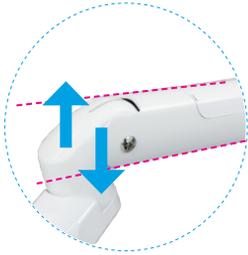

F. By using 2nd joint, you can rotate it left right.



G. By using 1st joint, you can rotate it 360°.

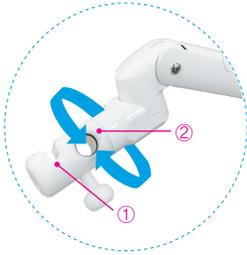

H. Start suction at your desired position.



5. Precautions

5. Precautions

1. Do not use the product if it is not fully punctual.
2. Replace the damaged part immediately if you spot it.
3. Do not make the product transformed.
4. Dry the product if you use it in wet condition.
5. Disinfect the product immediately if you find the product contaminated.
6. If repair needed, please call a merchant to receive the proper after service.

6. Product Specifications

- Contact your local service provider if any problem or malfunction has been occurred.
- Manufacturer : Megagen Global Co. Ltd.
- Address : 45, Secheon-ro 7-gil, Dasa-eup, Dalseong-gun, Daegu, Korea
- Tel : 82-53-222-2828 / Fax : 82-53-289-3420

Model Name	2nd Assi
Intended Use	It is used to hold the suction instrument while treatment.
Packing Unit	1 SET

*** This product is a medical device.**

Seoul Office Megagen Tower 4F, 607, Seolleung-ro, Gangnam-gu, Seoul, Korea T.1566-2338
Headquarter 45, Sechon-ro 7gil, Dasa-eup, Dalseong-gun, Daegu, Korea T.1544-2285

